

FLOCKED NYLON SWAB

Flocked swabs have brought tipped applicators to a new level. these swabs bring superior collection and release characteristics to labs

- ✓ 6" long shaft enables easier sampling at difficult sites without direct exposure to the user
- ✓ Short perpendicular nylon fiber strands attached to flexible molded plastic results in efficient collection and release of particulate matter
- ✓ Recent references have show better sample yield with flocked swabs than conventional swabs
- ✓ Higher sample yield maximizes sensitivity of cell culture and molecular techniques
- ✓ Flexible plastic shaft delivers better patient comfort



Nasopharyngeal Swab

Insert dry swab in to nostril and back to the nasopharynx. Leave in place for a few seconds. Slowly withdraw the swab with a rotating motion.

Nasopharyngeal Sample Collection

Feather touch
Nylon Flock (PW1172)


Maximum
patient comfort

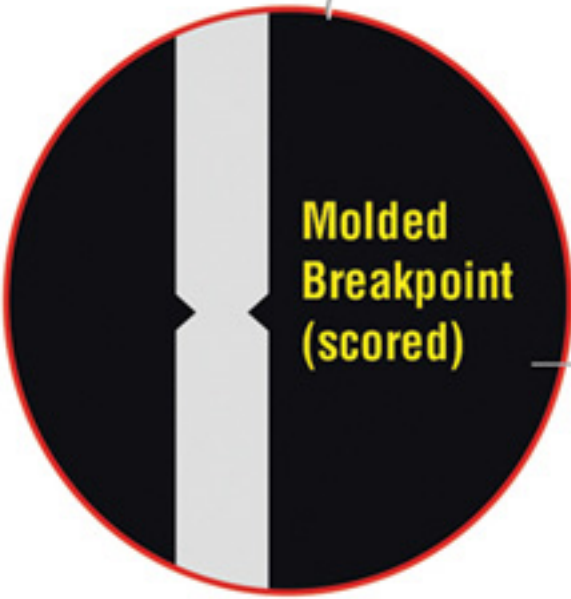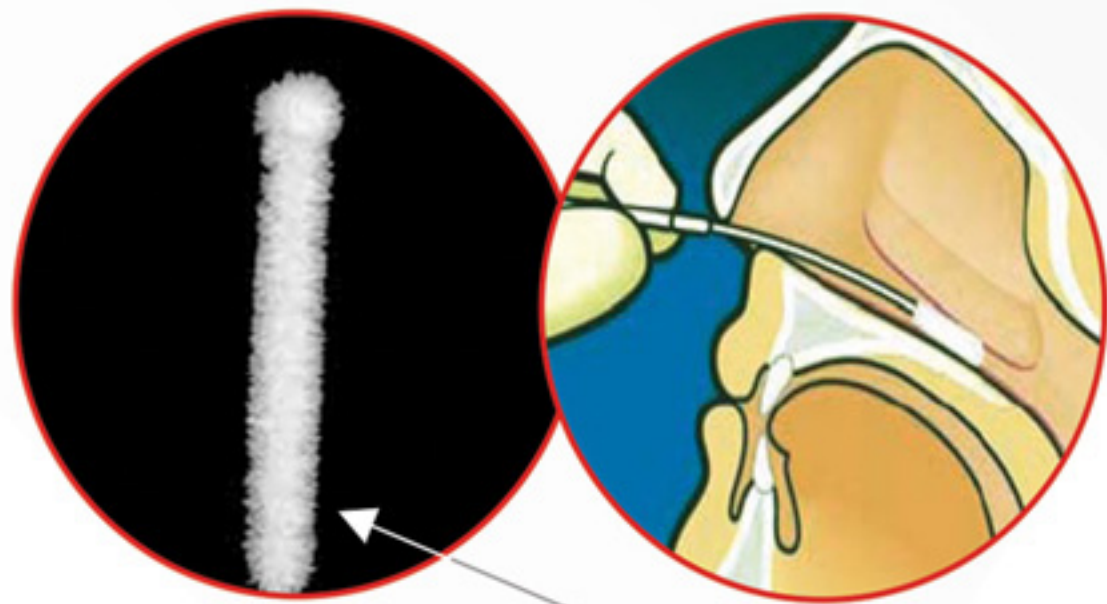

Throat Flocked Nylon Sample Collection




Throat Swab

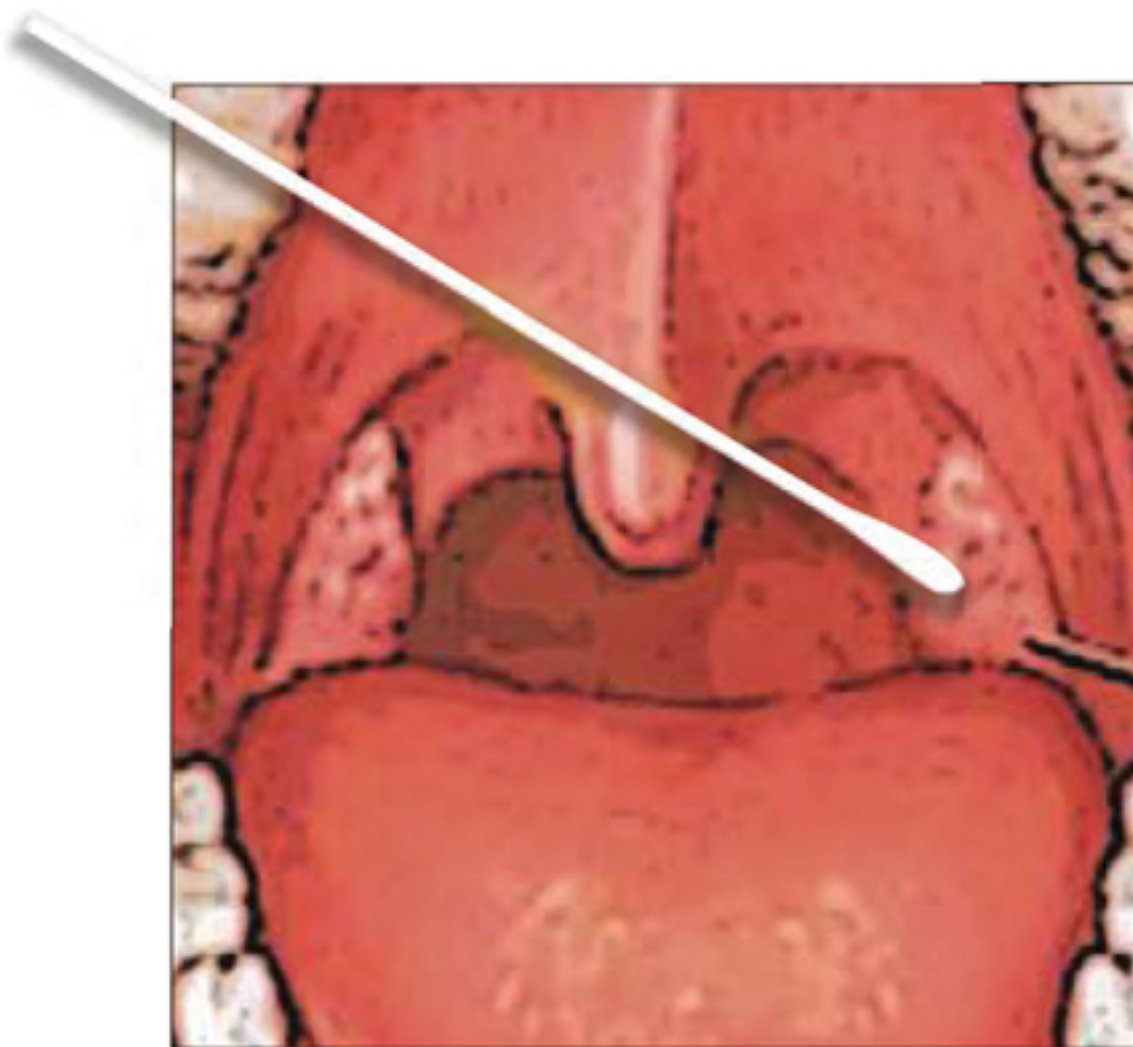

Throat Swab

Ask patient to open his/her mouth. Swab the back of the throat near the tonsils thoroughly.

Suggested Throat sample collection area
(Sample may be collected from other areas of the throat as per user established protocols)

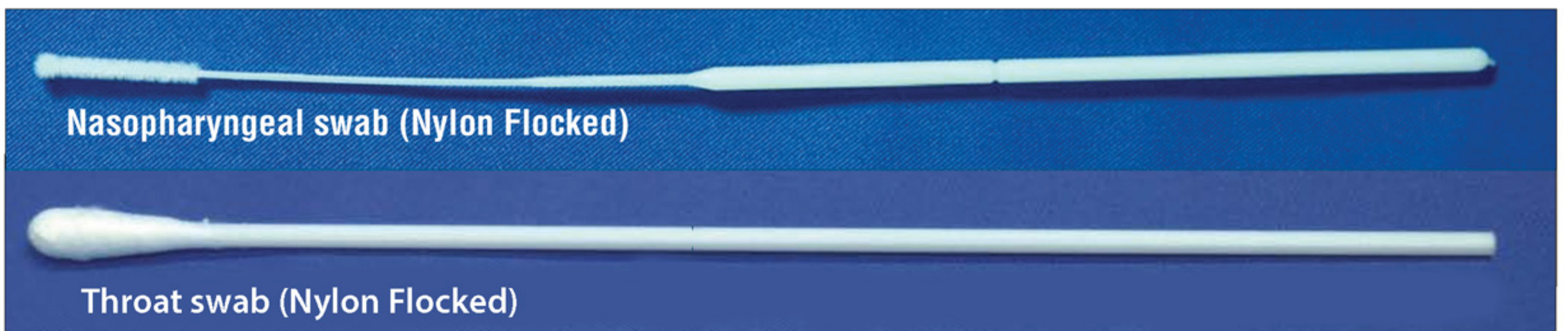

CE Marking



Recombigen Laboratories Pvt. Ltd.
98, Phase-II, Badli Industrial Estate
Near Sector 18, Rohini
Delhi - 110042, India



Do not use if package is damaged



+91-9811478785



officerecombigen@gmail.com



www.recombigen.com